



## 擺位枕 Nano Positioning Pillows

輔助提供舒適的姿勢，而適當擺位提供特定照護。

Nano Positioning Pillows increase comfort and adapt to the patients easily, giving more possibilities of positioning.

**APEX**

製造廠/藥商  
 雅博股份有限公司  
 23679 新北市土城區民生街9號  
 Apex Medical Corp.  
 No. 9, Min Sheng St., Tu-Cheng,  
 New Taipei City, 23679, Taiwan  
 T +886 2 2268 5568  
 F +886 2 2268 6526  
 customer@apexmedical.com.tw  
 marketing@apexmedicalcorp.com  
 消費者服務專線 0800 054 133  
 內容僅供參考，本公司保有規格及設計變更之權利

www.apexmedicalcorp.com

Distributed by

Print-2020/11-1000

All rights reserved. Specifications may change without notice

Care for a Healthy Life

**APEX**



### Positioning Ranges



- 


▶ **頭枕 Universal (Large) 60 x 41 cm**  
Allows head positioning aligned with the body with improved occipital area support
- 


▶ **小枕頭 Universal (Small) 32 x 21 cm**  
Avoid compressing the bony prominences (Ex: elbows, arms and shoulders)
- 


▶ **圓柱枕 Cylindrical (Large) 70 x 18 cm**  
Help leg support and relief pressure of the heels
- 


▶ **S枕 Decubitus (Large) 180 x 39 cm**  
Supports perfectly the body, from head to feet

APEX Nano Positioning Pillows come with a removal cover and also a zipper for easy cleaning and disinfection. Washable, vapour permeable, water resistant and antimicrobial allow its use in a wide range of healthcare facilities. Thorough drying is essential for the long life and good use of the material. They help reduce friction and shear, thus reducing the risk of development of pressure injuries.

維持臥床者的功能位置，讓身體獲得最大面積的支撐，以降低接觸面壓力；新姿勢擺放5-10分鐘後，檢查皮膚易受壓力的部位，每2小時調整病人位置。

#### 平躺(仰臥) Supine



#### 側躺(側臥) Lateral



**頭枕 Universal (Large)**  
用頭枕支撐頭部 Head supported with a pillow

**小枕頭 Universal (Small)**  
手臂下方擺放枕頭，避免壓迫骨凸點 (肘、肩、髖)  
Place under the arm to avoid compressing the bony prominences. (Ex: elbows, arms and shoulders)

**S枕 Decubitus (Large)**  
上方膝蓋微彎，錯開膝蓋  
不要互疊  
Place between the knees

**S枕 Decubitus (Large)**  
S枕靠在背上，將身體保持在適當的位置  
Place snugly against the back to maintain in a proper position

**S枕 Decubitus (Large)**  
足踝內側不要壓迫  
Don't press the inside of ankle

**圓柱枕 Cylindrical**  
雙膝伸直，圓柱枕置膝蓋下方  
Place underneath the knees

-  Fire Resistant 防火材質
-  Water Resistant 防水
-  Breathable 透氣
-  High Elasticity 高彈性
-  Biocompatibility 生物相容性
-  Treated with Silver Ions 銀離子抗菌



#### 產品規格

外枕套: 銀離子+彈性布 Silver+ Stretch / 內枕套: 棉質 Cotton / 內容物: 發泡聚苯乙烯 Expandable Polystyrene